



## Installing the Sentinel System Driver Installer


1. When you run Sentinel Protection Installer 7.x.x.exe, Install Wizard is activated. Please click 'Next'.

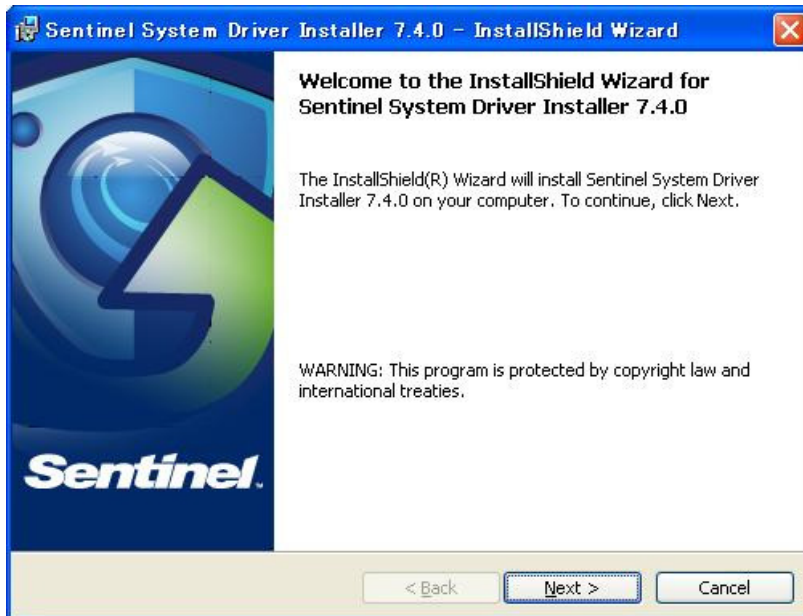

If an old version of the System Driver is already installed, the screen below will appear.

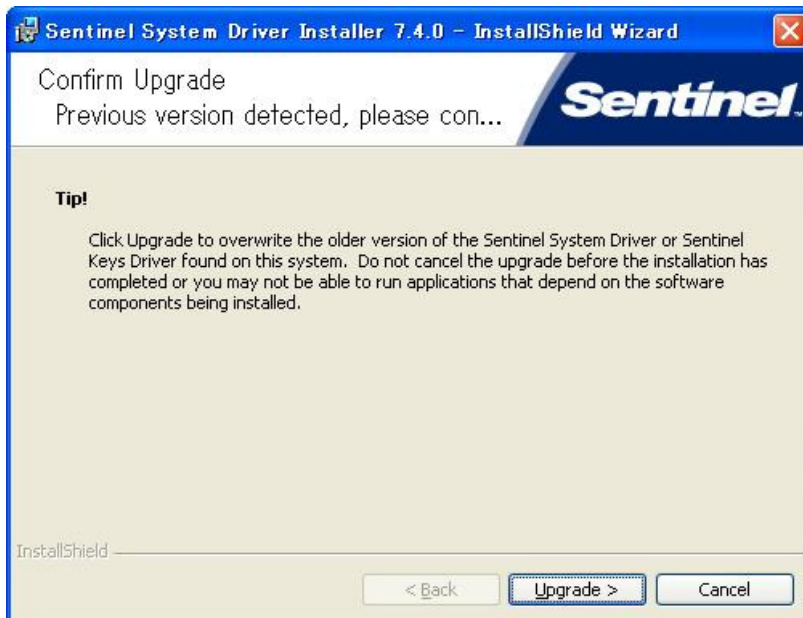


2. Licensing agreement information will be displayed. To agree to its terms and continue, please click 'I accept the terms in the license agreement' and click the 'Next' button. If you do not select 'I accept' you will not be able to click the 'next' button.

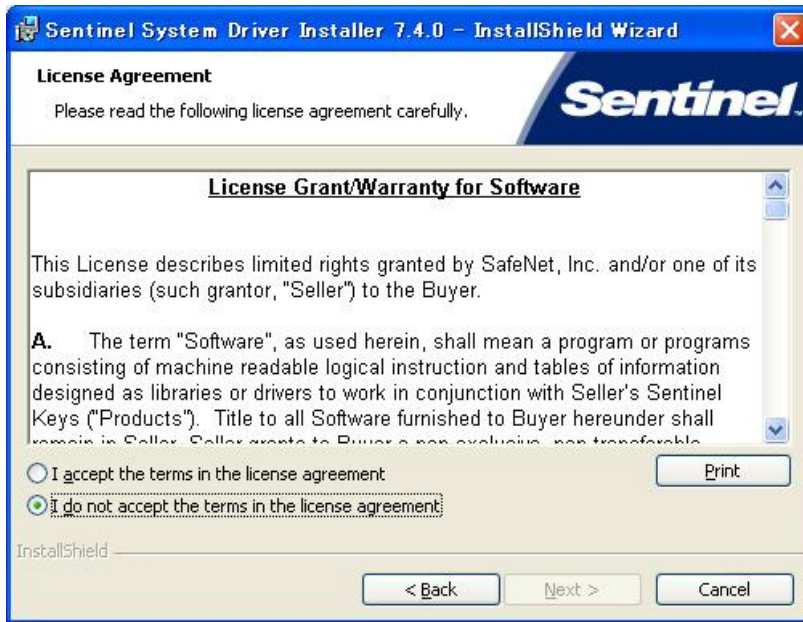


3. Please choose one of the following 2 options and click 'Next.'

- Complete: Install all the component parts of the System Driver Installer.
- Custom: User selects the desired component parts and the user can select the default destination for installed components.

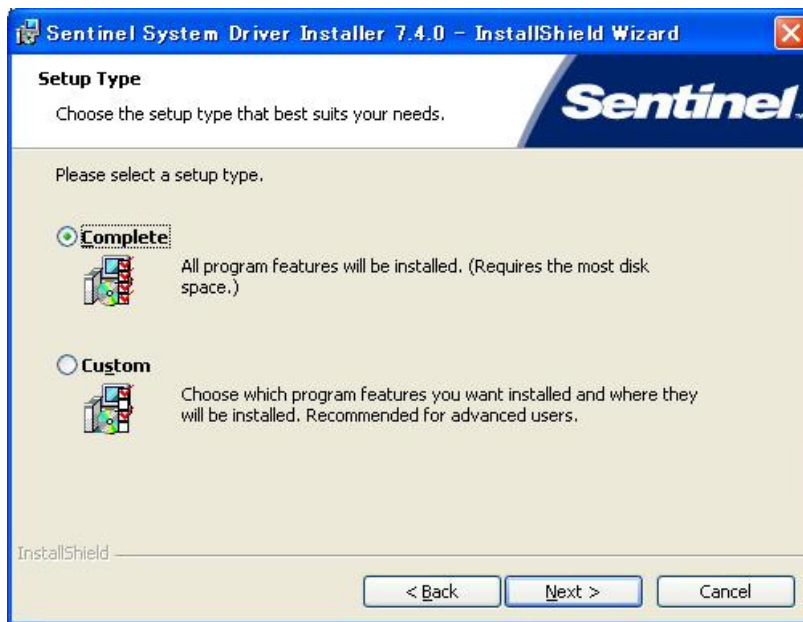


If 'Complete' is selected, the program moves on to step 5. If 'Custom' is selected, a list of all component parts is displayed. Please proceed to the next stage.


4. If 'Custom' is selected, the following screen is displayed. You can check off the components that you wish to install as a default.

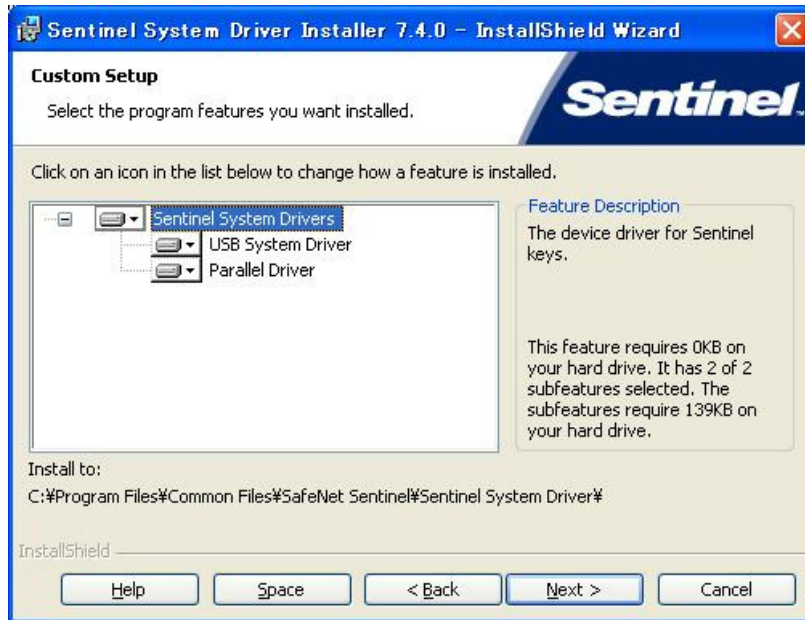

5. When the 'Ready to Install the Program' screen is displayed, Please click 'Install'

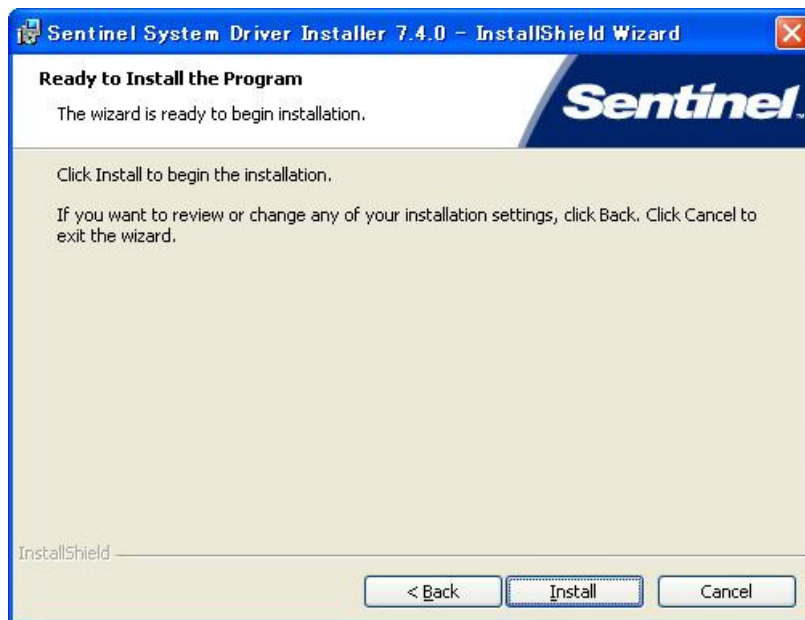

6. Start installation.

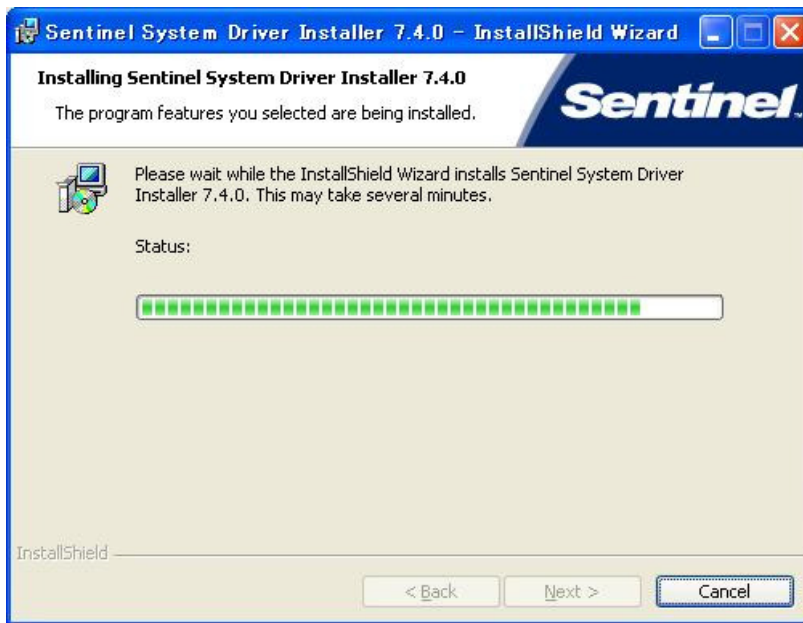

7. When installation is completed, the 'Install Complete' screen is displayed. Please click 'Finish' to exit the wizard.

